

CODAN Infusion management



CODAN Drip SWAN® and CODAN Guard®

Wing-Valve ▼

The Wing-Valve at the top end of the drip chamber is used to prepare, fill and also simply aerate and ventilate the system whilst medication is being administered. During administration, air collects in the drip chamber and the liquid level is reduced correspondingly. In order to correct the liquid level, the drip chamber can be refilled to the normal level by pressing on the wings of the valve.

CODAN Drip SWAN® and CODAN Guard®



CODAN Worldwide

CODAN is known internationally as a manufacturer and supplier of disposable medical transfer systems. The CODAN Companies have more than 1500 employees around the world.

The name CODAN is synonymous with reliability, quality and precision based on the know-how and experience gained from more than 60 years of research and development. Company-owned production facilities and sales companies around the world are a guarantee for efficient production, a tight-knit sales network and a first-class service for customers in the healthcare sector.

Complete administration of the medication without rest volumes



▲ SWAN-LOCK®

Thanks to the SWAN-LOCK® component integrated into the drip chamber, medication can be added to infusions needle-free. So the Drip SWAN® is yet another CODAN Product which clearly helps increasing safety both for patients and for medical personnel, and which minimises any risk to the greatest possible extent.

Direct injection into the drip chamber ▼

In clinical settings, medical staff is increasingly relying on medication supplied in syringes. With the Drip SWAN® system, medication can be added directly, enabling it to blend with the carrier solution inside the drip chamber. Record of protocol should be carried out by the responsible persons at the pharmacies. A corresponding protocol concept is available from CODAN.

More time for patients and medical personnel

Where traditional sets have to be changed, the Drip SWAN® and CODAN Guard® systems can be used for a longer period. Accordingly, the Drip SWAN® and CODAN Guard® systems help reducing costs and waste while still ensuring that the medication is completely administered. Especially for antibiotics it is clear that a complete administration of the whole therapy is of great importance. The Drip SWAN® and CODAN Guard® systems have additional advantages with regard to storage and reduction in staff time required.



CODAN Guard®

The CODAN Guard® has been developed to provide an even more user-friendly infusion handling. This new generation of infusion sets is equipped with an AirStop filter membrane integrated into the drip chamber in order to prevent air infusion.



The user benefits from being able to change the infusion container/bag quicker, minimising the risk of contamination and air infusion. Rather than re-using a set which has already had air enter its tubing from where it would have to be eliminated, simply switch over to the next infusion container and re-prime the set, letting the administration continue effortlessly.

Complete administration

Do you want your patient to receive the whole medication and at the same time to have more safety for the patient and medical personnel?

CODAN offers you a clean, safe and simple solution with our new combination of infusion sets.


CODAN Companies

- CODAN Medizinische Geräte GmbH & Co KG · Deutschland
- CODAN pvb Critical Care GmbH · Deutschland
- CODAN pvb Medical GmbH · Deutschland
- CODAN PORTUGAL, S.A. · Portugal
- CODAN 11, S.A. · Portugal
- CODAN US Corporation · California · USA
- CODAN Inc. · California · USA
- CODAN NORGE AS · Norge
- CODAN TRIPLUS AB · Sverige
- CODAN Limited · Great Britain
- CODAN FRANCE Sarl · France
- CODAN Medical AG · Schweiz
- CODAN ARGUS AG · Schweiz
- CODAN BV · Nederland
- CODAN s.r.l. · Italia
- CODAN Medical GmbH · Österreich
- CODAN Steritex ApS · Danmark
- CODAN Medical ApS · Danmark
- CODAN DEHA ApS · Danmark

CODAN Product range

- Infusion sets
- Transfusion sets
- Extension lines and manifold connectors
- Infusion and transfusion accessories
- Infusion filters and filter systems
- Neonatology/Paediatric products
- Withdrawal, preparation and administration systems
- CODAN CYTO®
- Chemoprotect® products
- CODAN ENTERAL
- Single use syringes
- Invasive blood pressure monitoring systems
- Infusion pumps
- Continence care products
- Other CODAN Products

Compliance of the established quality management systems with the provisions of EN ISO 13485 and the Council Directive 93/42/EEC has been certified by the relevant, competent notified bodies. Our products are thus entitled to bear the following marks:

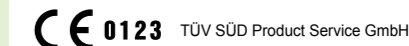

CODAN Medizinische Geräte GmbH & Co KG
CODAN pvb Critical Care GmbH
CODAN US Corporation
CODAN ARGUS AG



CODAN Steritex ApS
CODAN Medical ApS

Manufacturer
CODAN Medizinische Geräte GmbH & Co KG
Stig Husted-Andersen Straße 11 · 23738 Lensahn, Germany
Tel. +49 (0) 43 63 · 51 11 · Fax. +49 (0) 43 63 · 51 12 14
www.codan.de

The decisive connection

