


# CODAN DPT - 6000 system

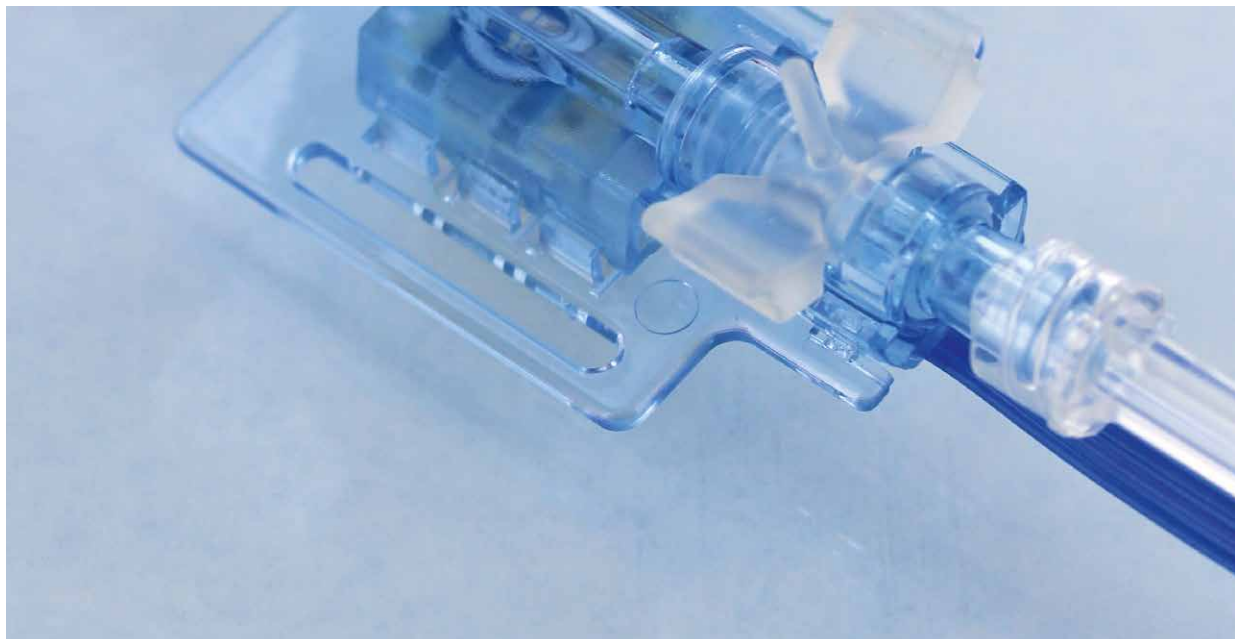

Precision and system diversity in invasive blood pressure monitoring




# CODAN DPT - 6000 system

## System diversity in invasive blood pressure monitoring

The pressure transducers in the DPT-6000 range combine all the features that hospitals require from an advanced and reliable transducer system for invasive blood pressure monitoring.

The simple handling of the system and its compact, ergonomic design ensure reliable operation.

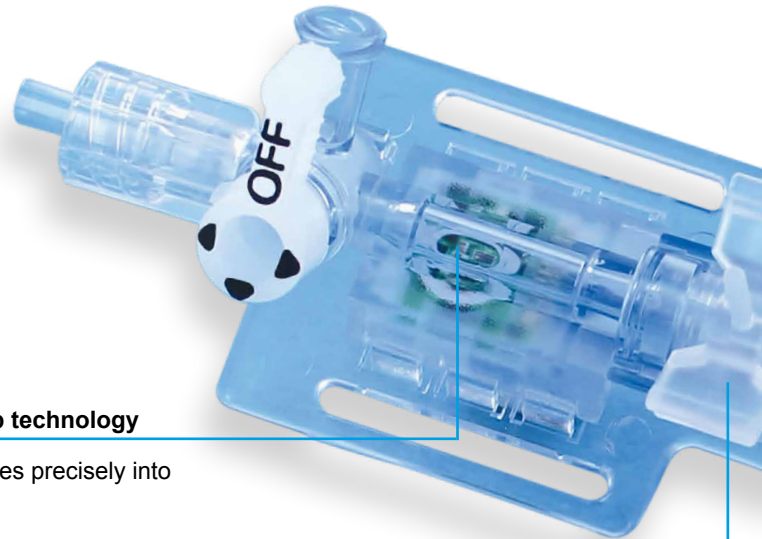


### **State-of-the-art microchip technology**

converts the actual pressures precisely into electrical signals.

### **The integrated dual-function flushing system**

with its linear flow allows rapid and bubble-free priming of the connected pressure monitoring set.

Squeezing the wings of the flushing device generates a high flow for rapid flushing, while the pull tab can be used to test the transducer's response. If a pressure increase (square waveform) is displayed undistorted on the monitor, the transmission system (tubing) has been correctly de-bubbled.

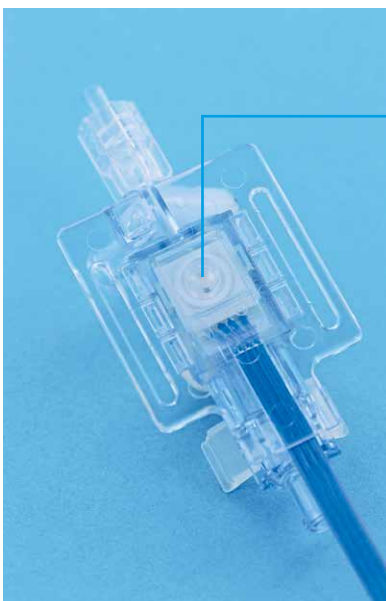

### **The Luer connector on the rear allows**

the Transducer Tester to be connected for active testing of the complete monitoring system, without compromising sterility. The test signal appears inverted on the monitor.


### **PVC-free transducer cable**

with special, splash-proof cable connector and transducer housing.

## Wide range of system accessories


### Interface cables for connecting the pressure transducers to the control monitor

These interface cables are individually tailored to suit the equipment of the particular hospital. The monitor end is fitted with the requisite monitor connector, while the patient end has a connector for the pressure transducer.

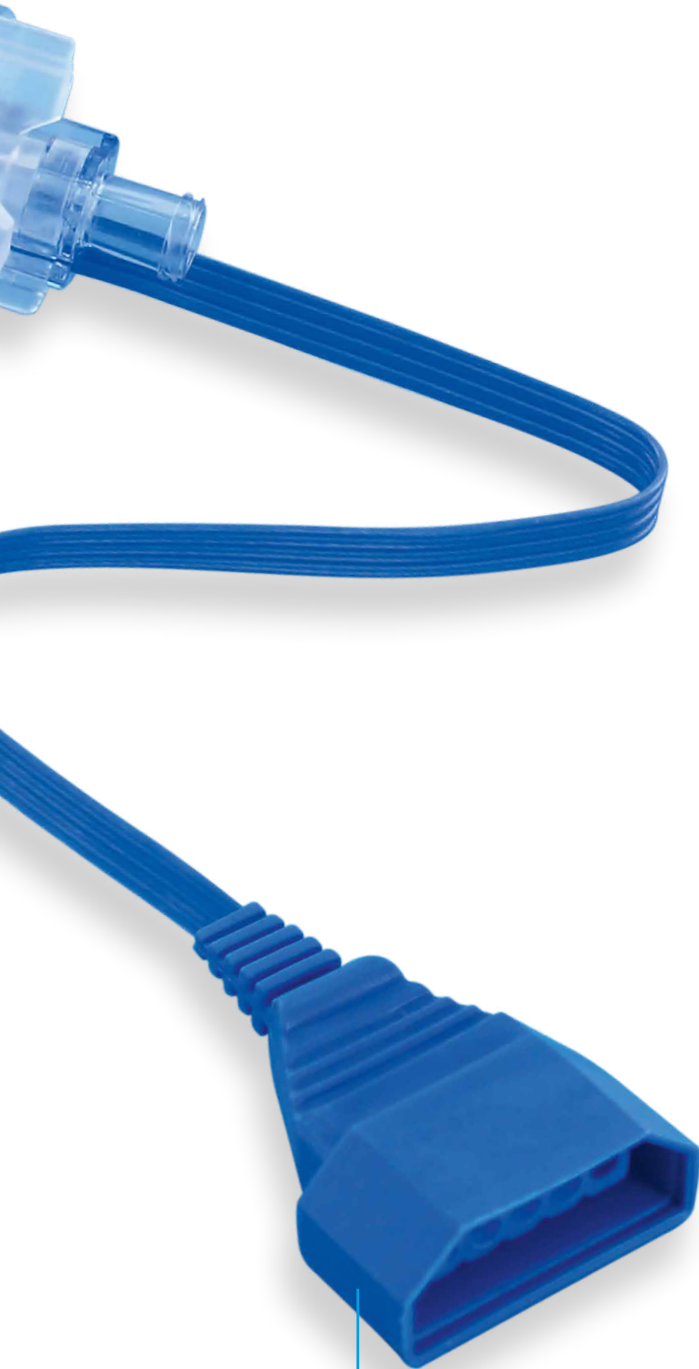

### Modular mounting plate system

This mounting plate is specially designed for DPT-6000 range of pressure transducers. The side channels allow the individual mounting plates to be interconnected as a modular system. The physician can thus combine as many of these mounting plates as required in each particular case. The mounting plate is made of a durable, robust plastic. It is reusable and washer safe.

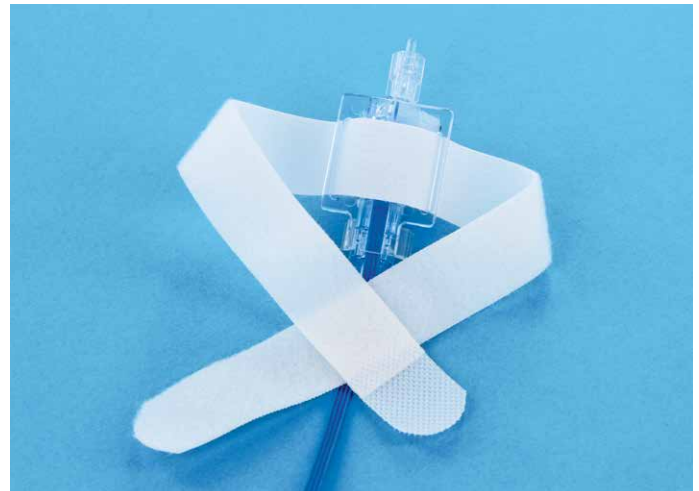

### Velcro strip for attaching the transducer to the patient's arm

Depending on the medical indication, it may be necessary to position the transducer as close as possible to the puncture site access.

This Velcro strip can be used to attach the transducer directly on the patient's arm, in which case very short pressure monitoring lines can be used.

# CODAN Worldwide



CODAN is known internationally as a manufacturer and supplier of disposable medical transfer systems. The CODAN Companies have more than 1500 employees around the world.

The name CODAN is synonymous with reliability, quality and precision based on the know-how and experience gained from more than 60 years of research and development. Company-owned production facilities and sales companies around the world are a guarantee for efficient production, a tight-knit sales network and a first-class service for customers in the healthcare sector.

## CODAN Companies

CODAN Medizinische Geräte GmbH & Co KG · Deutschland  
CODAN pvb Critical Care GmbH · Deutschland  
CODAN pvb Medical GmbH · Deutschland  
CODAN PORTUGAL, S.A. · Portugal  
CODAN 11, S.A. · Portugal  
CODAN US Corporation · California · USA  
CODAN Inc. · California · USA  
CODAN NORGE AS · Norge  
CODAN TRIPLUS AB · Sverige  
CODAN Limited · Great Britain  
CODAN FRANCE Sarl · France  
CODAN Medical AG · Schweiz  
CODAN ARGUS AG · Schweiz  
CODAN BV · Nederland  
CODAN s.r.l. · Italia  
CODAN Medical GmbH · Österreich  
CODAN Steritex ApS · Danmark  
CODAN Medical ApS · Danmark  
CODAN DEHA ApS · Danmark

### Manufacturer

CODAN pvb Critical Care GmbH  
Römerstraße 18 · 85661 Forstinning, Germany  
Tel. +49 (0) 81 21 · 98 02 0 · Fax. +49 (0) 81 21 · 98 02 666  
codan@codanpvbccc.de · www.codan.de



**The decisive connection**



### CODAN Product range

- Infusion sets
- Transfusion sets
- Extension lines and manifold connectors
- Infusion and transfusion accessories
- Infusion filters and filter systems
- Neonatology/Paediatric products
- Withdrawal, preparation and administration systems
- CODAN CYTO®
- Chemoprotect® products
- CODAN ENTERAL
- Single use syringes
- Invasive blood pressure monitoring systems
- Infusion pumps
- Continence care products
- Other CODAN Products

Compliance of the established quality management systems with the provisions of EN ISO 13485 and the Council Directive 93/42/EEC has been certified by the relevant, competent notified bodies. Our products are thus entitled to bear the following marks:

### CE 0123 TÜV SÜD Product Service GmbH

CODAN Medizinische Geräte GmbH & Co KG  
23738 Lensahn, Germany

CODAN pvb Critical Care GmbH  
85661 Forstinning, Germany

CODAN US Corporation · Santa Ana, CA 92704, USA  
CODAN ARGUS AG · 6340 Baar, Switzerland

### CE 0543 Presafe Denmark A/S

CODAN Steritex ApS · 3060 Esbjerg, Denmark  
CODAN Medical ApS · 4970 Rødby, Denmark